



The following is a list of electrosurgical generators that can be used with the ClearGlide Precision Bipolar and bipolar cables.

Generator	Recommended Power Settings	Mode
Pegasys	15 – 20 Watts	Bipolar
Erbe ICC350	35 – 40 Watts	Bipolar
Valleylab Force FX	30 – 35 Watts	Bipolar Standard
ConMed ExcaliberPLUS	30 – 35 Watts	Bipolar Cut
ConMed Sabre 2400	30 – 35 Watts	Bipolar
Valleylab Force 2	30 – 35 Watts	Bipolar
Valleylab Force 4B	15 – 20 Watts	Bipolar Standard
Wolf 2352.011	20 – 25 Watts	Bipolar



Sorin Group USA, Inc.
 14401 W. 65th Way, Arvada, CO 80004
 800.221.7943 • 303.467.6525 fax
www.soringroup-usa.com/EVH

©2009 Sorin Group USA, Inc.

Caution: Federal law (USA) restricts this device to sale by or on the order of a physician.
 Trademark used herein are owned by or licensed to Sorin Group USA, Inc.

Part number 306490096 Rev B (04/09) Printed in U.S.A.